

Breast Wear

Size Chart & Measuring Instructions

This size chart is intended for use with the following breast wear styles: 450, 452, 454, 454-1, 455-V, 455-VN, 457Z, 457, 458, 458U, 456, 465, 4510.

Design Veronique® bra sizes do not conform to industry standard bra sizing. If you are having a breast augmentation, reduction, or reconstruction, your exact cup size will not be determined until after the procedure. All the measurements on our size chart are important. Please make sure to follow the measuring instructions.

MEASURING INSTRUCTIONS:

To ensure a proper fit, please have someone else take your measurements and avoid wearing bulky clothes. Your feet should be hip width apart and your arms relaxed at your sides. We recommend using a cloth measuring tape. Hold the tape level and firmly, but do not allow it to “dig” into your skin or clothing.


Under Bust Measurement: Before surgery, measure around the chest, directly beneath the breasts using the diagram on the right as a guide. This is your under bust measurement. Select the size from the size chart that matches your under bust measurement.

Sizing Example: An under bust measurement of 31” is a size M.

Cup Size: Design Veronique® bras are uniquely designed to fit your breast size after surgery, no cup size measurement is required. All Design Veronique bras are based on your under bust measurement.

Measuring and fit tips...

- Under bust measurement is required to determine your accurate size.
- Bra extenders, which increase the circumference of the band, are available for all Design Veronique® bras.

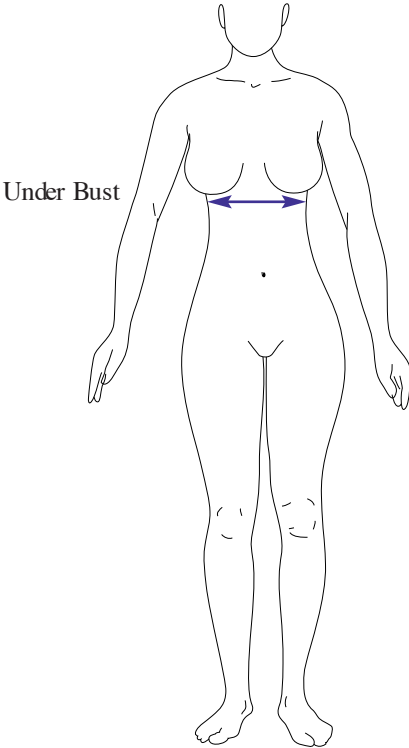

Match your under bust measurement to the chart below to find the appropriate size.

Size (inch)	Under Bust Measurement
XS	25" - 26"
S	27" - 30"
M	31" - 34"
L	35" - 38"
XL	39" - 42"
XXL	43" - 46"
3X	47" - 49"

Web Links

- Design Veronique® Web Page
- FAQ
- Garment Care
- Full Size Chart